# Instructions Manual

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**Commercial trade name:** Actif for institutions

**Registered name:** Actif Age, Lda

**Intended use:** To offer cognitive and physical stimulation activities for institutionalised older adults, through videos and recommendations at our web platform <https://pro.actif.online>. The product is intended for use by professionals in healthcare institutions, nursing homes or residences.


**Expected clinical benefits:** Healthier ageing through the prevention of cognitive decline and frailty syndromes.

### Recommended Use:

Physical activity (PA) - Ideally it should be practised 3 times a week, for 150-300 minutes of moderate PA, or 75-150 minutes of vigorous PA, which should include strength and balance training.

Cognitive training (CT) - Ideally, it should be practised 3 times a week, in sessions of 30 to 60 minutes.

In both cases, continuous practice within the recommendations enhances the overall results. These recommendations may need to be adjusted for each individual, based on their exercise capacity and specific health risks or limitations.

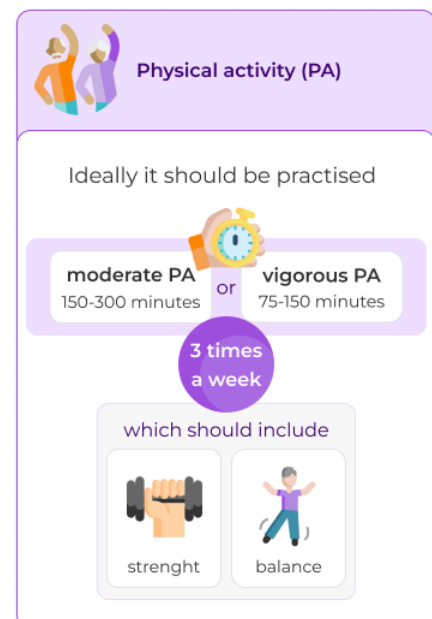**Device performance:** Unlimited access to physical activity and cognitive training videos with different intensities. The platform offers personalised recommendations and plans, and records each user's progress, after the implementation of scientifically validated assessment scales.

**Contraindications and risks:** It is mandatory that the institutional staff monitors the users during the activities.

In users with cardiovascular diseases, metabolic diseases (such as Diabetes Mellitus) or chronic renal failure, a medical consultation is advised prior to the initiation of device use, even if their disease is stable and asymptomatic.

For generally healthy individuals, moderate exercise is safe. If inactive, begin with lower intensity and progress in duration and intensity over time. If inactive initially, a routine clinical evaluation is recommended. If already active, medical clearance is recommended before engaging in vigorous activities.

This device is contraindicated for users with unstable angina, uncharacterized arrhythmias or decompensated heart failure.

This device is contraindicated for people diagnosed with epilepsy or other neurological disorders, due to the possibility of seizures.

This device is contraindicated in people with severe musculoskeletal injuries.

The benefit of using this device may be reduced in cases of hearing and visual impairment or cognitive impairment.

At any time, if the user experiences physical pain, dizziness, nausea or any other discomfort, they should stop the activity immediately, and be examined by a healthcare professional before resuming the activity.

**Specifications for use:** The product should primarily be used by the professionals that will assist the users, and must register on the platform: user data; individual assessment of user performance and well-being after activities; and record of their evolution.

The planning of activities must be done by the professionals, but can be suggested by the platform.

Activity preparation must include a comfortable and safe space, such as: a spacious and suitable place for the activity; suitable atmospheric conditions, avoiding excessive heat, cold and humidity; adequate lighting; appropriate and

safe materials, such as support equipment and footwear; calm space, without noise or distractions.

The presence of one or more professionals during the activities is mandatory.

**Warnings:** Actif Age Lda is not responsible for the consequences of the inappropriate use of its products, services or solutions. Its general recommendations are based on scientific knowledge, but its individual application must be advised and managed by health qualified professionals. In case of an incident, contact the manufacturer via email at [support@actif.online](mailto:support@actif.online), and the competent authority.

### Requirements

Internet access is required to device use.

The platform is compatible with:

Smart TVs

Computers

Tablets

Smartphones

**Requirements:** Internet access is required to device use. The platform is compatible with Smart TVs, Tablets, Computers or Smartphones.

**IT Security:** Access data to the platform is personal and non-transferable. If there is illegitimate access to data, users may lose their historical progress and recommendations on the platform, as well as incur third-party data privacy violations. To minimise these risks, Actif Age Lda carries out periodic privacy policies reviews, frequently updates the technologies in use, uses only stable and tested new releases, and uses selected and reliable technologies. Despite this, we alone cannot guarantee the complete security of your account, it is very important that you remain alert. To this end, we warn you that:

- Never share your password with anyone
- Do not leave your device unattended if you are logged in
- Do not use low security public networks
- If you suspect someone has accessed your account, reset your password immediately
- Use different passwords for each platform